

Mediox

Bone Cement Mixer and Pusher

Mediox Bone Cement Mixer and Pusher

Features and Advantages :

- Way of mixing bone cement


Compared with traditional way of mixing bone cement in bowl, this set allows bone cement to mix directly in the tube, which avoids bone cement splashing around and delay the bone cement hardening progress so as to allow enough surgery time.

- Simplify surgery steps

No need Syringe to draw bone cement. Instead, this set allows bone cement to be injected directly right after mixing, which simplifies the process and save the surgery time.


- Measurement on tube easily and accurately shows how much bone cement is injected. Easier for surgeon to calculate the bone cement dosage.

Components:

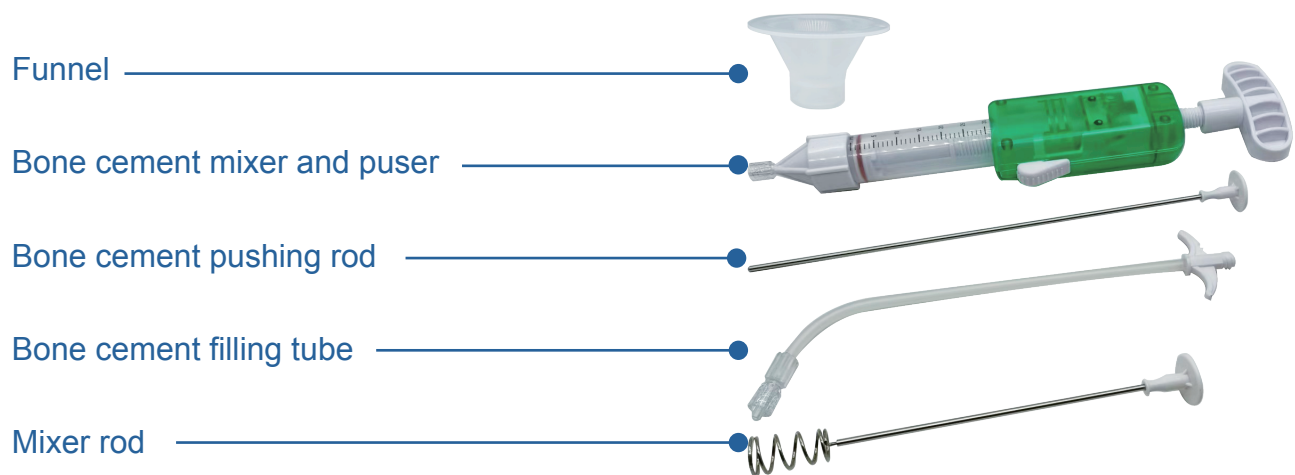

Surgical steps:

1) Push the white buttons and pull the handle as far as possible. Pour the powder and liquid through Funnel.

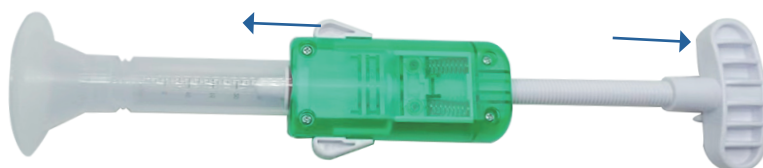

2) Mix the powder and liquid.



3) Remove the Mixer rod. Assemble the Bone cement filling tube.



4) Inject the bone cement.

5) Remove the Bone cement filling tube. Use Bone cement pushing rod to push the last drop of bone cement left in Bone cement filling tube.



Product Information:

Product Name	Product Code
Bone Cement Mixer and Pusher	S771752500





MEDIOX Orvosi Műszergyártó Kft

3324 Fels tárkány, 2473 Hrsz. HUNGARY

Fax: 0036 36 431 132

Tel: 0036 36 431 132